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## URGENT FIELD SAFETY NOTICE

### HALO VEST

### PRODUCT ENHANCEMENT – IMPROVED CPR ACCESS

April 10, 2017

#### AFFECTED DEVICES

Össur Halo Vest	Lil Angel Halo Vest
545700L	545100K
545800L	545100L
545700K	545100S
545800K	545150K
545700S	545150L
545800S	545150S

#### ATTENTION

Össur is committed to providing safe, high-quality medical devices to its customers.

A halo vest naturally limits the accessibility to the chest in the event of an emergency situation, therefore we felt it important to send a safety alert reminding our valued Össur customers of this risk. This alert explains how to properly remove an Össur vest, as well as any additional actions required.

In an effort to improve visibility of product removal instructions in the event of an emergency, Össur has redesigned the emergency CPR access labeling for the Össur Halo and Lil Angel Vests. In addition to improved visibility, the new label uses symbols that can be universally understood.

#### ACTION

Please ensure all relevant personnel are trained in the proper removal of the Halo and Lil Angel Vest in the event of an emergency. Refer to the included CPR instruction documents for Halo and Lil Angel for training purposes.

REPLACEMENT EMERGENCY CPR ACCESS LABELS

New Adult Vest Label	
Existing Adult Vest Label	
New Pediatric Vest Label	
Existing Pediatric Vest Label	 <p data-bbox="788 1375 916 1429">Sticker loose in bag</p>

This notice is to be communicated to all those within your organization, and to any other organization where affected devices may have been provided or serviced.

Please maintain awareness of this notice and recommended actions.

Please contact customer service on this phone number for further information and assistance:  
Tel: +44 (0)8450 065 065

*Hulda Halgrímsdóttir*  
Hulda Halgrímsdóttir  
Vice President, Quality & Regulatory