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<Date of Letter Deployment>

GEHC Ref# 12223

To: Hospital Administrators/Risk Managers
Managers of Radiology/Cardiology
Radiologists/Cardiologists

RE: **Discovery IGS 730 and Discovery IGS 740 systems - Potential unintentional motion or rotation due to positioning user Interface (Smart Box).**

GE Healthcare has recently become aware of a potential safety issue with respect to a potential unintentional system motion or rotation due to a positioning user interface issue with your Discovery IGS 730 & 740 products. **Please ensure that all potential users in your facility are made aware of this safety notification and the recommended actions.**

Safety Issue

The affected systems may experience an unintentional motion or rotation due to a positioning user interface issue. The potential reversed motion of the gantry/table has been observed when the user positions the joystick at its maximum position (end of travel) on the Smart Box. The unintended reversal of the direction of the motion may lead to a collision of the table or C-Arm with medical instruments, patient or personnel, and result in injury. There have been no injuries reported due to this issue.

Safety Instructions

If you observe any unintentional motion or rotation during the use of the Smart Box joystick control on your Discovery IGS 730 or Discovery IGS 740 product, immediately release the control/joystick to stop the motion.

Important Note: You can continue to use the system normally **but you must remember not to push the joystick of the Smart Box to its maximum position.** The collision avoidance and sensors are fully functional on your system and are not impacted by this issue.

Affected Product Details

Discovery IGS 730 & Discovery IGS 740 Systems with affected system ID's:
00169VAS01; 10692VAS01; 190020RX26; 203694IGS730; 610954OR1; A4185526
M4160476; S41600101; YV2000; 718283D740; M54866AG1; SO4208228; SO4242294
XR649425BU6; HC4348XR11; 206VMDISCOVERY1; 330375HOR; M40480222;
190059RX30; AE1477RX04; SO4239583; SO4165355; 82416120043; 82416040061.

Product Correction

GE Healthcare will correct all affected systems with a software upgrade at no cost to you. A GE Healthcare service representative will contact you to arrange for this correction.

Contact Information

If you have any questions or concerns regarding this notification, please contact the following phone number or email:

UAE Service Center (phone): 8003646 (Landline : +97144296162)

UAE Service Center (email): UAEServiceCenter@ge.com

You can also contact your local GE Healthcare Service Representative.