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Insulin pump Infusion sets

من انتاج الشركات: Medtronic of Canada Inc., Roche Diagnostics,

LifeScan Canada Inc., and Auto Control Medical Inc.

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الشركة	نوع الصنف
Medtronic of Canada Ltd.	Paradigm Sure-T Infusion Sets Silhouette Infusion Sets Paradigm Silhouette Infusion Sets
Roche Diagnostics	Accu-Chek Tender I Accu-Chek Tender II
Auto Control Medical Inc.	Comfort Short Infusion Sets Comfort Kit Infusion Sets
LifeScan Canada Ltd	Comfort Short Infusion Sets Contact Detach Infusion Sets

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Urgent safety information for insulin pump users

Report a Concern

Starting date:

November 21, 2014

Type of communication:

Information Update

Subcategory:

Specialized Products

Source of recall:

Health Canada

Identification number:

RA-42629

November 21, 2014

For immediate release

OTTAWA – Unomedical a/s, in consultation with Health Canada, is informing Canadians and Canadian healthcare practitioners that there is a potential safety issue regarding infusion sets distributed by [Medtronic of Canada Inc.](#), Roche Diagnostics, LifeScan Canada Inc., and Auto Control Medical Inc.

Unomedical a/s, the manufacturer of these infusion sets, has identified an increase in reports of the tubing becoming detached at the connect/disconnect location on the products. Patients who are concerned about their device should contact their distributor.

If tubing detachment occurs, insulin delivery is interrupted and the pump alarm will not notify the user, according to the manufacturer. The risk to patients of an interruption in insulin delivery is hyperglycemia (elevated blood sugar levels). Hyperglycemia can result in diabetic ketoacidosis, which is a serious condition, and in extreme cases can lead to death. Symptoms of diabetic ketoacidosis may include nausea, vomiting, shortness of breath and excess thirst/urination.

The manufacturer is advising patients to continue to use the infusion sets while following the directions below:

- When changing your infusion set, closely follow the instructions for use included in the product box. Check the tubing at the site connector location to make sure it is not loose.
- Monitor your blood sugar levels frequently using your blood glucose meter. Proactively check your tubing connections occasionally to ensure tubing is secure. It is especially important to check your blood sugar and tubing connections at bedtime to confirm insulin delivery is occurring.

- If you experience high blood sugar, check your tubing connections and infusion site closely to ensure your tubing is secure.

If you discover that the tubing is detached:

- Do not attempt to reattach the tubing. Replace the infusion set immediately.
- Treat any high blood sugar based on guidelines provided by your healthcare professional.
- Call your distributor (see table below) to report the issue and receive instructions on how to return the affected infusion set to the company.

Distributor List

Company	Model	Reference Number	Contact Information
<u>Medtronic of Canada Ltd.</u>	Paradigm Sure-T Infusion Sets	MMT-862	1-800-646-4633
		MMT-864	
		MMT-874	
	Silhouette Infusion Sets	MMT-371	
		MMT-373	
		MMT-377600	
	Paradigm Silhouette Infusion Sets	MMT-378600	
		MMT-381600	
		MMT-368600	
		MMT-383600	
		MMT-384600	
Roche Diagnostics	Accu-Chek Tender I	04541430001	1-800-688-4578
		04541413001	
		04541448001	
	Accu-Chek Tender II	04541464001 04541472001	
Auto Control Medical Inc.	Comfort Short Infusion Sets	89-060-2522	(905) 814-6350 x 252 1-800-461-0991 x 252
		10000600 10000601 10000602 10000603	
LifeScan Canada Ltd	Comfort Kit Infusion Sets	10024001	1-877-937-7867
		10024003	
		10024004	
		10024005	
		10090500	
	Contact Detach Infusion Sets	10090501 10090502	

Distributor List

Company

Model

**Reference
Number**

**Contact
Information**

10090503